DDS Filippo Cardarelli
Dentist
Specialist in Orthodontics
University of Milan

1st Class Deepbite

9-year-old patient deep bite dental and skeletal. Malocclusion is characterized by contraction of the upper arch with reduced space for eruption of permanent teeth.

The therapeutic sequence involves the following steps:

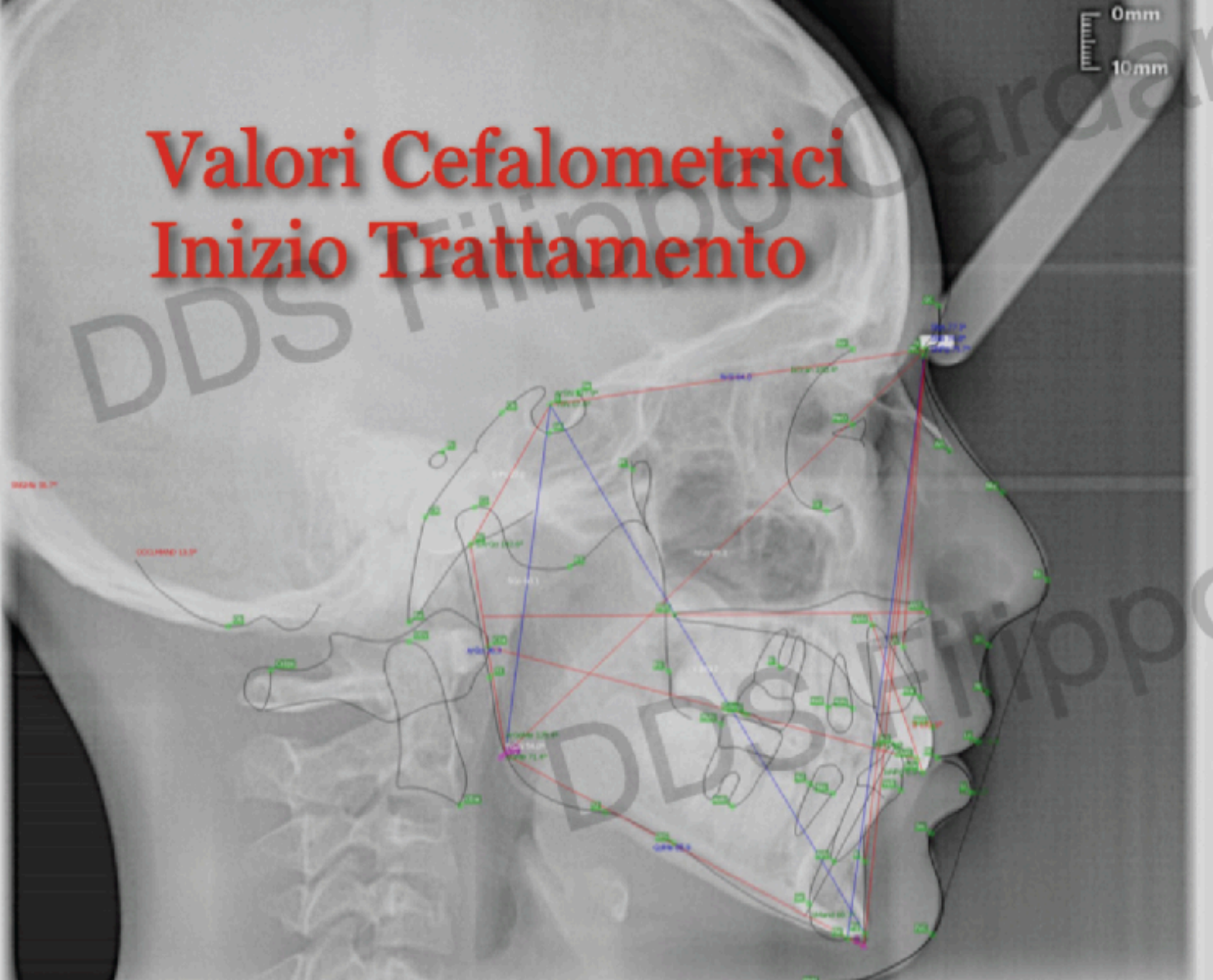
- Motivation
- Elastodontic therapy (duration 14 months)

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LED

1st Step

The elastodontic appliance used was the AMCOP Integral with the purpose of expanding the upper arch and coordinating the arches.

Use was one hour during the day and every night for 6 months and then for another 8 months only at night.

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
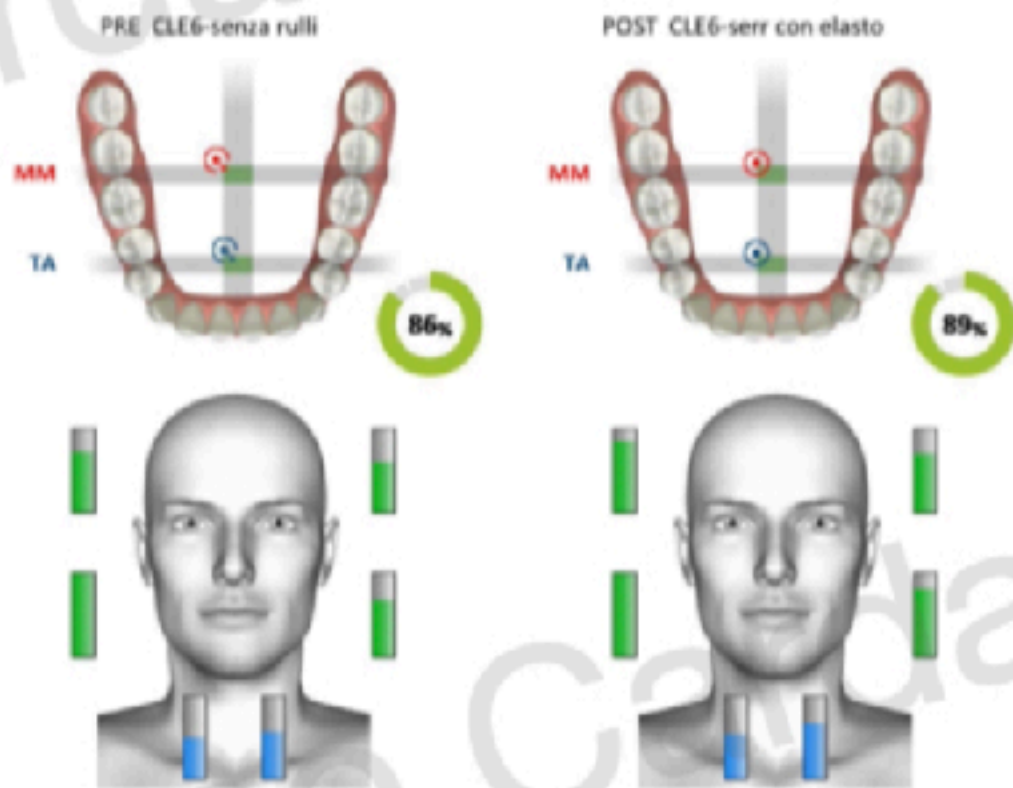
INDICI	PRE		POST		NORMALITA'
POC TA	87.36%	Dx	85.45%	Dx	83s(%)s100
POC MM	79.68%	Dx	85.46%	Dx	83s(%)s100
BAR	87.66%	P	90.26%	P	90s(%)s100
TORS	89.69%	Sx	91.45%		90s(%)s100
IMP	191.83%		150.28%		85s(%)s115
ASIM	14.53%		9.60%		-10s(%)s10
POC SCM	86.48%	Sx	85.77%	Sx	83s(%)s100
CL	3.95%		3.57%		0s(%)s15

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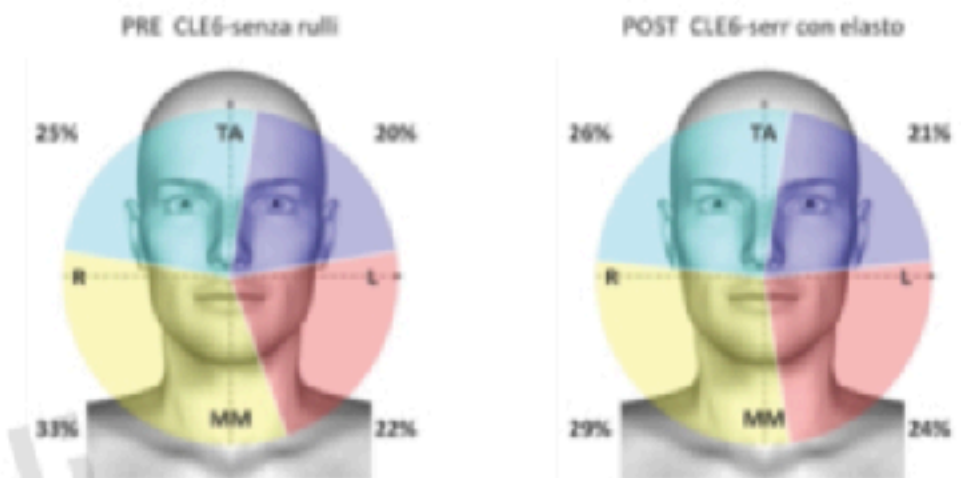BTS TMJOINT REPORT

Conclusions

The deep bite malocclusion turns out to be corrected with the Amcop elastodontic appliance with Integral mastication plan, as is evident from the radiographic and photographic documentation but more importantly, at the end of the therapy an excellent muscle balance was obtained as can be seen from the final electromyographic examination.

DDS Filippo Cardarelli